



wordofmouth  
technology

## Mobile Phone Accessories



The Solution for using Hearing Aids with Mobile Phones



Available for most Mobile Phones, the **T-Link** allows the wearer the convenience a portable handsfree that couples with the persons hearing aid using the “T” or Telecoil switch.

Adaptor may be required for some models.

The **T-Link** is a simple solution to allow hearing aid users interference free access to most mobile phones as well as other communications.

Formed from high grade black Nylon, the **T-Link** is profiled to fit behind the ear and at less than 2 mm thick, it leaves plenty of room for BTE hearing aids as well as glasses. In use the **T-Link** looks and works in an identical manner to regular hands-free kits with a high quality in-line microphone to be sure that you are heard clearly.



Using the audio systems within the mobile phone, the **T-Link** requires no batteries or supporting components - just plug it in—and that's all there is to it! When using T-Link keep the mobile phone away from your head, place it on your belt or on a table. The **T-Link** works by isolating the radio frequencies from the hearing aid and inductively coupling the audio signals to it when the “T” position is selected. If your hearing aid does not have the “T” switch, it can in most cases be upgraded.

**T-Link** is available from [Word of Mouth Technology Pty Ltd.](http://www.wom.com.au)

RRP for the Universal T Link is \$89.00 GST Free.

RRP for the Binaural T Link is \$110.00 GST Free